
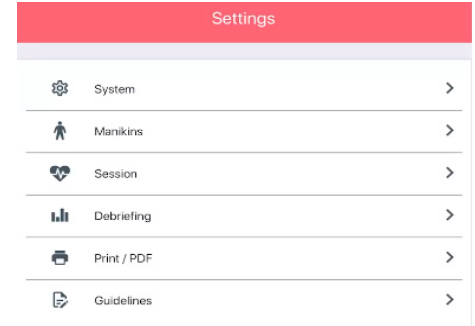


Due to the high demand for feedback to improve the results of BLS manikins, Ambu provides high quality feedback. **iQF – Integrated Quality Feedback**. Designed for trainees and instructors, the iQF application offers intelligent tools to improve the outcome of training with great precision for the performance of infants and adults in cardiac arrest.

In these instructions you will find out how to adjust the settings as an instructor.

1. CONNECT THE END DEVICE WITH THE TRAINING DEVICE

To connect the device and the manikin, turn on the manikin, activate bluetooth and the location. Then click on the instructor's app. To access the setting click on the set-up in the bottom of the screen  Here you see the overview of the six taps setting available.



2. ADJUSTING SETTINGS

Click on "System Settings"

If you want to change the **language** by clicking on the dropdown. You can select or deselect if the manikin should enable automatic data.

You can also choose if you want to have "**Metric or non Metric**" display.



If a **factory reset** is needed then click on the black button in the bottom of the screen.

Click on "Manikins"

Here all connected manikins are shown, to enter click on "Inspect". you will find the name, the type, ID and software. The **name can be changed** by clicking in the name field. Click enter once typed .

Power safe mode is available, this means the manikin will be turned off after 30 mins without use.

Click on "**Update Firmware**" to automatically update the manikin to the latest firmware. A notification will appear, stating the firmware was successfully updated. Disconnect the manikin and restart.

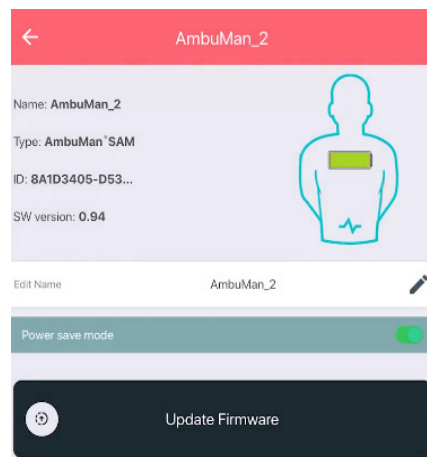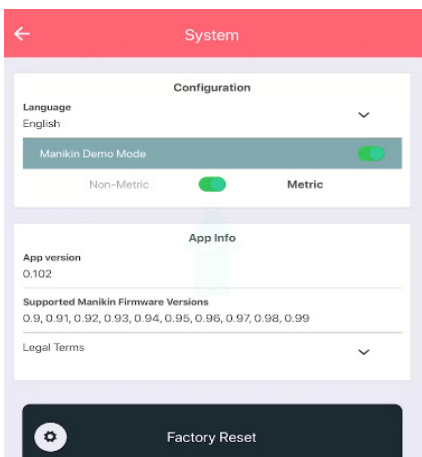

Click on "Session"

To configure the session. The main screen here shows the **overview** of a session. Up to four values can be selected.



The "**Detailed**" view allows you to select the set of values that are available during the session without limitations.

If you are having a line graph or bar graph diagram, you can go to the third tab "**Graphics**" where you can change the interval of the screen.

Click on "Debriefing"

To select which values you want to have shown.

The "**Detailed**" view allows you to select the set of values that are available during the session without limitations.

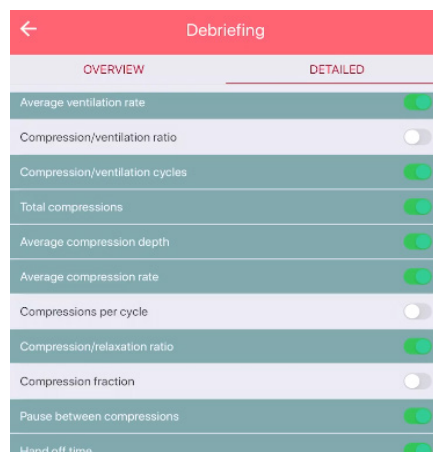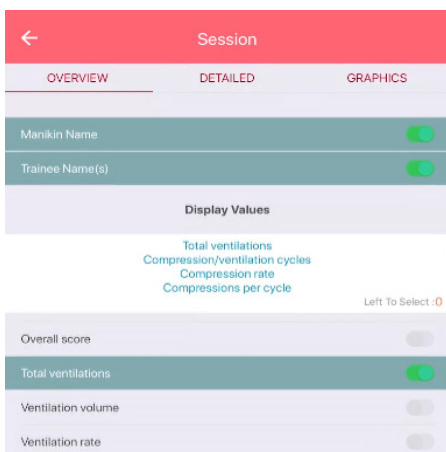


How to adjust the settings as an instructor

Click on "Print / PDF"

To decide how you want the printout to look.

Here you can able or desable the **Line Graph or the Radar Graph**.

If you click on the "Values" section you can decide which values should be shown in the printout or the PDF.

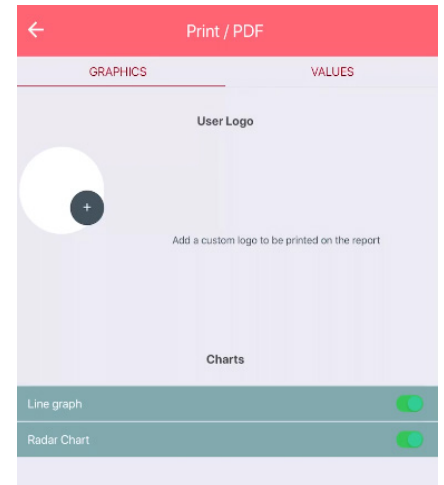


Click on "Guidelines"

To see the guidelines that are currently installed.

If you want to **view** them, click on them and if you want to modify something, just click inside. Change the value, confirm with OK and press the Disk button to store it.

The guidelines can also be **exported, imported and customized** by clicking on the desired option at the bottom.

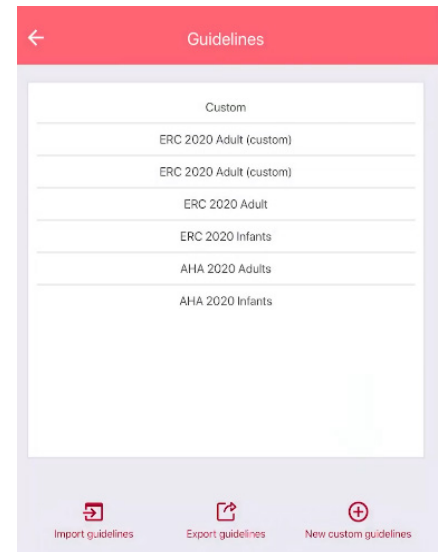

Quick and easy, the invincible mix!

Ambu

Ambu A/S
Baltorpbakken 13
2750 Ballerup
Denmark
T +45 72 25 20 00
F +45 72 25 20 50
ambu.com